


## Using Join-In Pro with Communicator 5

Tobii-Dynavox Communicator 5 users can access Join-In Pro apps directly from their displays, and toggle between their page sets and Join-In Pro apps. Users can:

- Send messages, posts and comments they composed using Communicator 5 to the various Join-In Pro apps (e.g., Facebook, Instagram, Whatsapp, Gmail, and more).
- Use Communicator 5 to search the various apps (e.g., search contacts in Whatsapp, videos in Youtube, etc.).
- Translate text composed in Communicator 5 by using the Join-In Pro Google Translate.

## Integrating Join-In Pro in Communicator 5

[C5 Join-In Pro Integration Template](#) – this page set includes all the buttons required for the integration. The page set contains three pages:

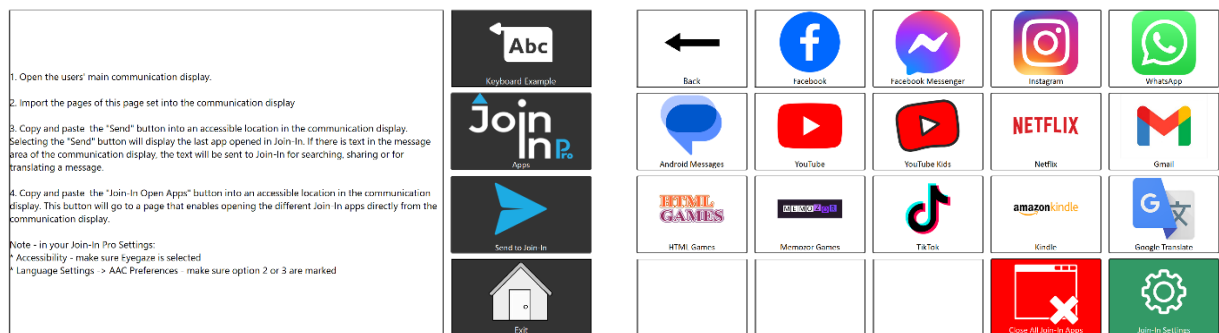

**Main page:** the **Send to Join-In** button switches to the last used Join-In Pro app and sends text from the Message Window to the app. Copy this button to your Communicator 5 Page Set.

Note: If Join-In Pro is closed, the button opens Join-In Pro App Explorer.

**Join-In Apps Page:** each button opens the specified Join-In app. The Close All button closes all apps. The Settings button opens Join-In Pro App Explorer.

Import this page to your Communicator 5 page set or copy buttons separately.



**Keyboard Example Page:** this page shows an example of a keyboard layout with Join-In Pro.

A Google Translate button can also be added to translate the text in the Message Window.



You can also download the page set [C5\\_Qwerty10 Join-In Pro Integration](#) - a fully integrated keyboard page set.

**Before starting to work with Join-In Pro and Communicator 5, you need to configure Join-In Pro Settings. This is a onetime activity that can be done right after installing Join-In Pro:**

- Open Join-In Pro go to **Settings, Accessibility** and select the appropriate **Input Device**.
- Go to the **Settings, Keyboard, AAC Preferences** – and select the option you prefer. Option 2 – use Communicator 5 to compose messages, posts, comments and search phrases. Option 3 – use Communicator 5 for all input, including usernames and passwords.

**A typical flow – using Join-In Pro with TD Snap to compose a message:**

- (1) Open one of the Join-In Pro messengers from your Communicator 5 page set.
- (2) In the messenger, select a contact, and click “Type Message”.
- (3) After being transferred to your Communicator 5 page set, compose your message, using your keyboard.
- (4) Click **Send to Join-In**. This will take you back to the messenger. Review the message and send it.

To download Join-In Pro, go to <https://apps.d-bur.com/downloads-support/>

**Questions? Comments?**

**Feel free to contact us at [info@d-bur.com](mailto:info@d-bur.com).**

